Compact⁺

User Manual

OPN: COMP-WRLD Version 1
© 2007 Optelec, the Netherlands
All rights reserved

Optelec
P.O. Box 399
2990 AJ Barendrecht
The Netherlands
Phone: +31 (0)88 678 34 44
E-mail: info@optelec.nl
Internet: www.optelec.com

Contents

1. Introduction 4
 About this manual 4
 What’s in the box 5
2. Getting to know your Compact+ 6
 Top side 6
 Front side 7
 Right side 8
 Underside..... 9
3. Getting started..... 11
 Placing the batteries 11
 Turning the power on / off 12
 Using the magnification button..... 12
 Selecting a viewing mode 12
 Using the snapshot button 13
 Writing with the Compact+ 13
 Using the Compact+ handgrip..... 14
4. Power and battery use 15
 Charging the batteries..... 15
 System Auto Off 15
Appendix A: Safety..... 16
Appendix B: Technical information..... 17

1. Introduction

Congratulations on choosing the Compact⁺, a pocket-size electronic video magnifier designed by Optelec.

Optelec's pocket-size video magnifier enlarges printed text and images up to 10 times on a 4.3" wide screen display.

With its one button simplicity, controlling both magnification and viewing modes, the Compact⁺ offers an enhanced design that extends in operating comfort. It is designed for every day use at any location.

If you have any questions or suggestions concerning the use of this product, please contact your distributor or the Optelec headquarters using the contact information at the last page of this manual. Your feedback is greatly appreciated. We hope you enjoy working with your Compact⁺.

About this manual

This manual will familiarize you with the features and basic operation of the Compact⁺. Please read this manual carefully before using your device.

What's in the box

The Compact⁺ package contains the following:

- The Compact⁺ pocketsize video magnifier
- A detachable neck cord
- A protection and carrying pouch
- A power cable with power adapter and 4 plugs (EU, UK, US and AUS)
- The user manual and operation sheet
- A battery pack containing two rechargeable AA batteries


If any items are missing from your package, please contact your distributor.

2. Getting to know your Compact⁺

The unit is placed correctly in front of you when the display is facing up and the round magnification / mode button is at the right side of the unit.

Top side

The following figure shows the top side of the Compact⁺.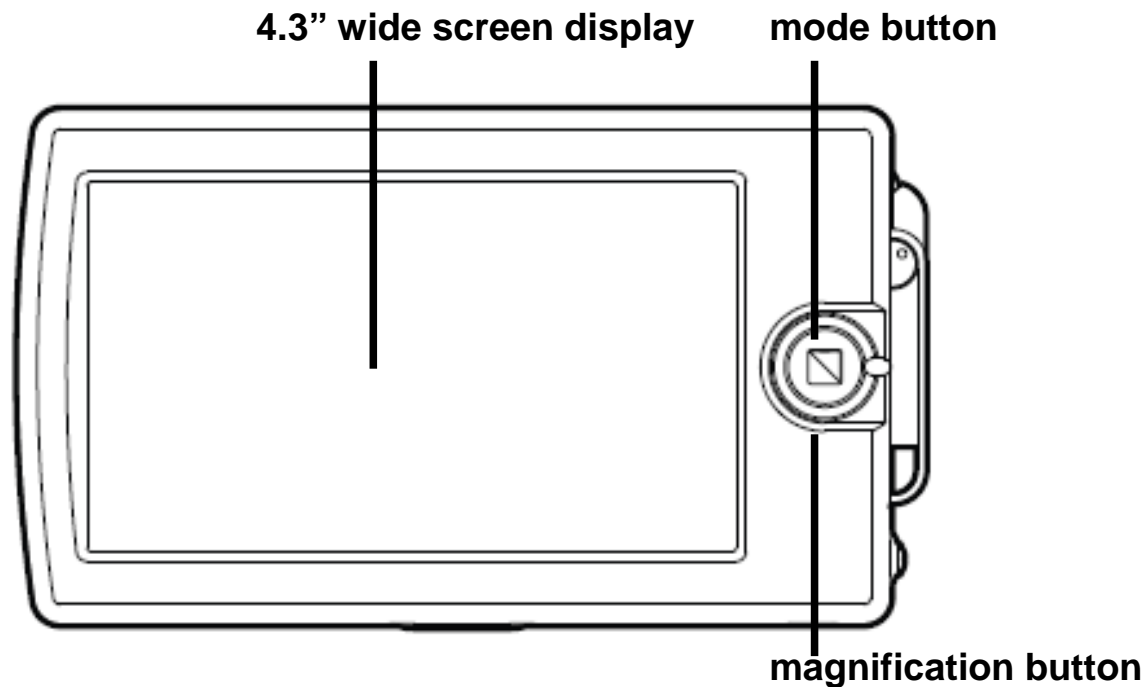

4.3" wide screen display

The 4.3" wide screen displays text and images

Mode button


Pressing the mode button allows you to select one of the six viewing modes

Magnification button

Turning the lever allows you to change the magnification

Front side

The following figure shows the front side of the Compact⁺.

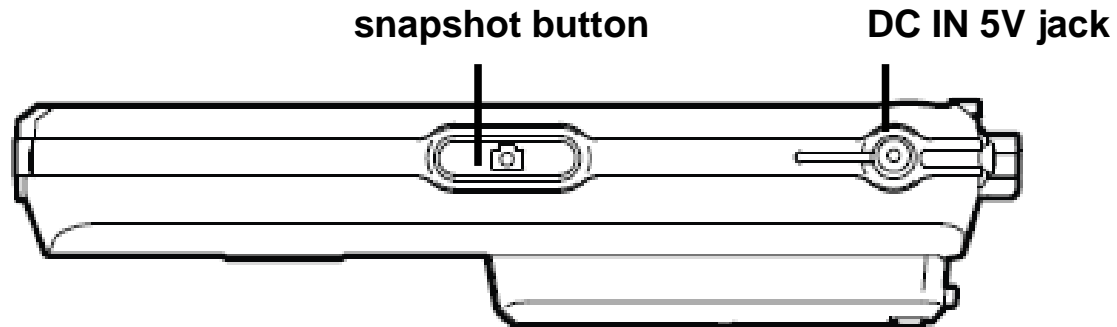**Snapshot button**


The snapshot button allows you to take a temporary picture of a distant text or image, enabling you to bring the image closer to your eyes

DC IN 5V jack

The AC adapter cable connects to this jack. Use only the model of AC adapter that comes with the Compact⁺.

Right side

The following figure shows the right side of the Compact⁺.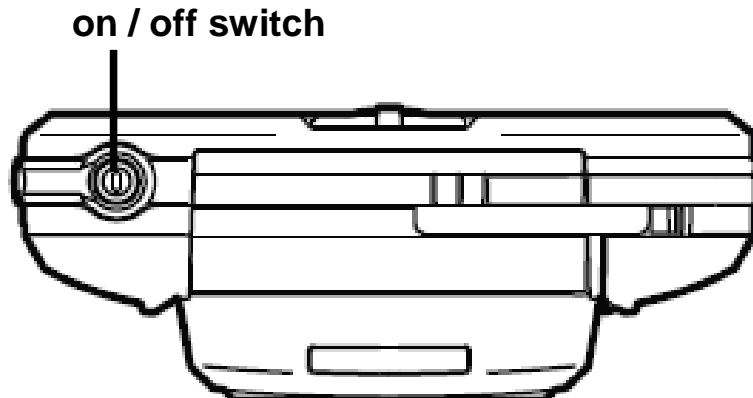


On / Off switch

Pressing the small round orange button for half a second will switch the unit on / off

Underside

The following figure shows the underside of the Compact⁺.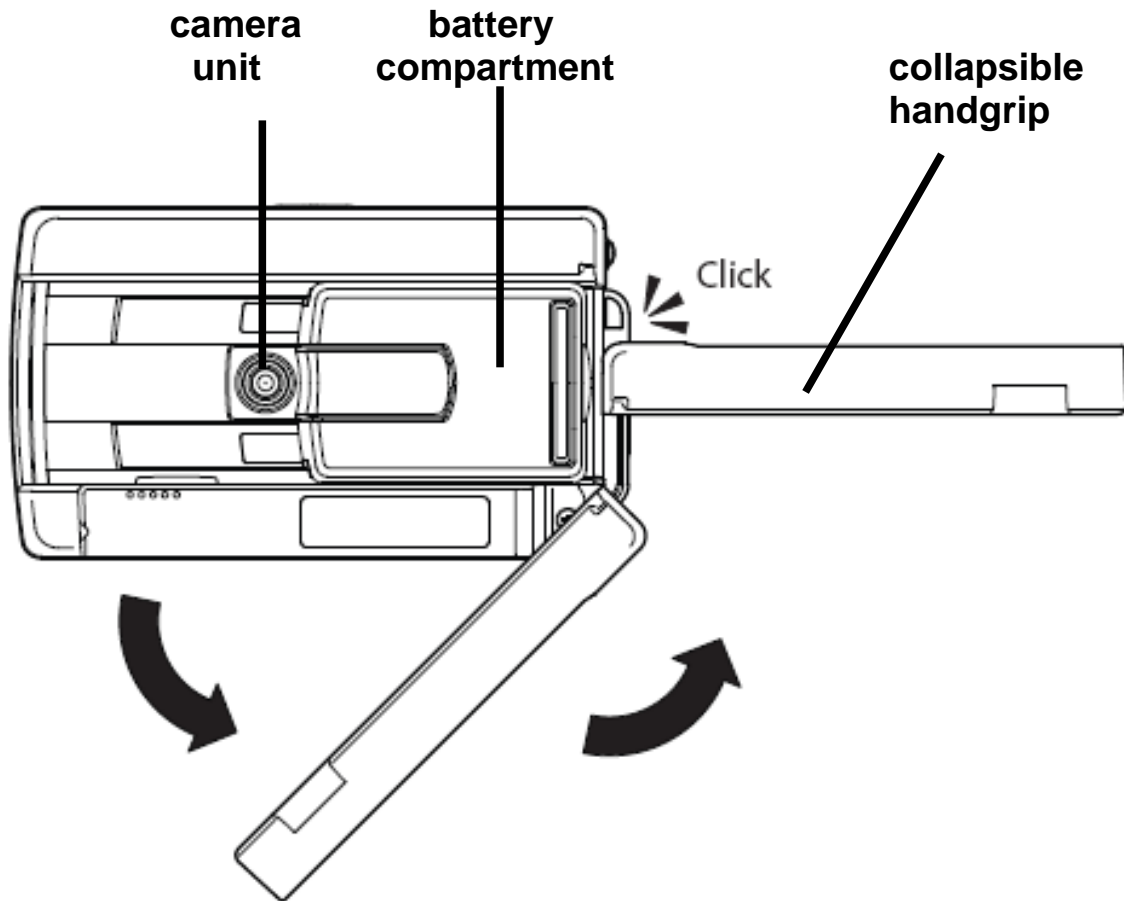

Battery compartment

To place the battery pack or two single AA batteries, either rechargeable or disposable

Camera unit

The Compact⁺ features an auto-focus camera that can slide in two positions; to be used for reading and writing

Handgrip


Pull and extend the collapsible handgrip for enhanced reading comfort

3. Getting started

Placing the batteries

The Compact⁺ operates on a battery pack consisting of two AA rechargeable batteries. Place the battery pack as follows:

- Turn the unit upside down
- Press the small half-oval tab at the far end to release the cover of the battery compartment
- Lift the cover
- Place the battery pack
- Set the cover on top of the battery compartment
- Gently press down to lock, until you hear a click

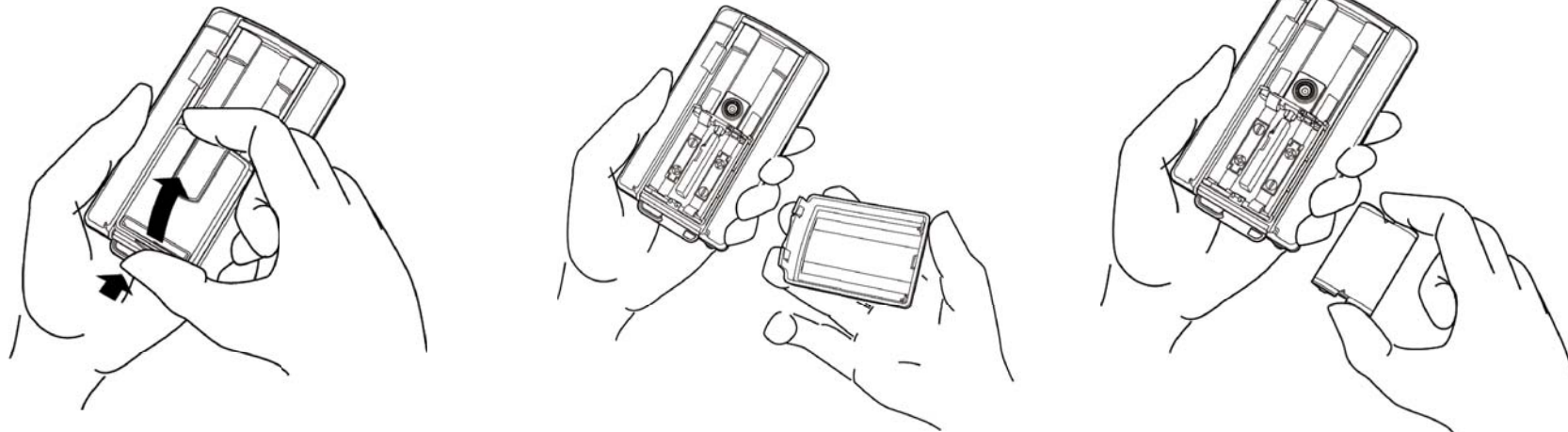Turning the power on / off

Switch the unit on and off by pressing the orange button, located at the right side of the unit, for half a second. By default the Compact⁺ will start in full color photo mode.

Using the magnification button

To set the magnification, simply turn the lever of the round button, located next to the display. Placing the unit on a flat surface, allows you to select one of the three magnification levels; 5X, 7.5X or 10X. Lifting the unit, allows you to lower the magnification.

Selecting a viewing mode

Pressing the round mode button, located next to the display, allow you to select one of the six viewing modes.

- Photo mode: view full color text and photos
- Positive mode: reading in high contrast black on white
- Negative mode: reading in high contrast white on black
- High contrast yellow text on a blue background: reading in high contrast yellow on blue
- High contrast yellow text on a black background: reading in high contrast yellow on black
- Photo mode without light: reading glossy text and displays

Using the snapshot button

The snapshot button allows you to take a temporary picture of a text or an image.

- Hold or place the Compact⁺ over the text or image
- Press the oval shaped snapshot button, located at the front side of the unit.
- Bring the image closer to your eyes to view the object or text
- Turn the lever, located next to the display, to change the magnification
- Press the round button, next to the screen, to change the viewing mode
- Pressing the snapshot button again will unfreeze the image

Writing with the Compact⁺

The Compact+ has an integrated writing facility.

Take the unit in your hand and turn it upside down (display facing down)


- Slide the battery compartment away from you (see picture)
- Put the unit back on your paper (display facing up)
- Press the Compact⁺ down on the side of the magnification / mode button
- Place your pen under the raised side and you can start writing

Slide the battery compartment to its original position to return to the reading mode.

Using the Compact⁺ handgrip

The Compact⁺ has a collapsible handgrip; giving you the feeling you are using a regular magnifying glass. Simply pull the handle, located at the underside, to release the handgrip and extend out to snap in place. The collapsible handgrip gives you extra support and control for optimal reading comfort.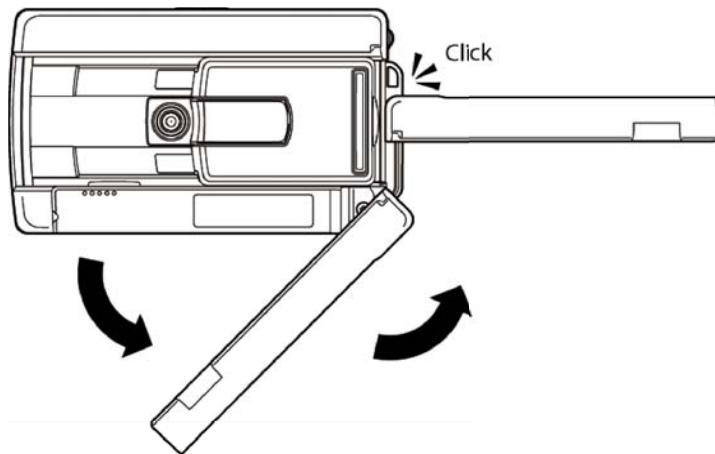

4. Power and battery use

When the AC adapter is not connected, the Compact⁺ main power source is a battery pack of two rechargeable AA batteries. Off-the-shelf AA batteries can also be used, either rechargeable or disposable. Out of concern for the environment and superior power, we advise you to use rechargeable batteries.

Charging the batteries

To charge the batteries connect the adapter supplied with the unit to the DC IN 5V jack, located at the front side. Plug the other side into a power outlet. The orange LED indicator next to the adapter socket will glow continuously while the battery pack is charging. When the battery pack is fully charged the LED light will switch off. If the LED indicator starts to flash, it means there is a charge problem. Disconnect the adapter, check if the battery pack is placed correctly in the compartment and try again.

Charging the Compact⁺ takes approximately 3 hours. Once the batteries are fully charged, you can use the unit for at least 3 hours continuously. Please note that, for safety reasons, only the batteries that came with the Compact⁺ can be charged in the unit's battery compartment.

System Auto Off

This feature will turn the Compact⁺ off automatically if it is not used for a period of 4 minutes. Pressing the orange button at the left side will switch the unit on again.

Appendix A: Safety

- Do not expose the Compact⁺ to excessive heat or direct sunlight to avoid the risk of fire.
- Do not remove any parts of the Compact⁺. Contact an Optelec authorized service organisation in your area.
- To avoid risk of electrical damage, keep your Compact+ away from fluids and chemicals.
- Handle the Compact⁺ with care. Rough handling will damage the internal components.
- Do not use the Compact⁺ near inadequately shielded medical devices.
- Contact your dealer to service this equipment if necessary. Do not open your unit as it will void the warranty.
- Always unplug the system before cleaning. Use a soft damp cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.
- Use only the supplied power adapter.

Use of the Compact⁺ other than described in this manual will exclude it from warranty conditions.

Appendix B: Technical information


Magnification	5x - 7.5x and 10x (on a flat surface)
Viewing modes	Full color photo mode High contrast black on white High contrast white on black High contrast yellow on blue High contrast yellow on black Photo mode without light
Focus	Auto-focus
Screen	4.3" widescreen TFT
Screen resolution	480 x 272
Battery	3 hours continuous use Charge time 3 hours
Weight	300 grams / 10.6 ounces (excl. adapter)
Power supply	120-240 VAC / 50-60 Hz 5V - 2 A.

Operation and storage conditions:

Relative humidity: <70 %, no condensation

Temperature: 0 – 40 ° C

Note : Used batteries should be disposed at a depot that is designated by local competent authorities.

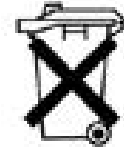WARNING: The use of accessories, transducers and cables, with the exception of transducers and cables sold by the manufacturer of the Compact⁺ as replacement parts for components, may result in increased emissions or decreased immunity of the Compact⁺.

CE, FCC and UL certification pending

Compact⁺

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OPN: COMP-WRLD Version 1
© 2007 Optelec, the Netherlands
All rights reserved

Optelec
P.O. Box 399
2990 AJ Barendrecht
The Netherlands
Phone: +31 (0)88 678 34 44
E-mail: info@optelec.nl
Internet: www.optelec.com

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Optelec Limited
Tieman GmbH